

Guidelines for using the **LM** Extraction Instruments

- The **LiftOut** and **SlimLift** are designed for gentle luxation only.
- The **TwistOut** is thicker in design and used when more force is needed.
- Sterilize all instruments before use and re-use. Max. 134°C/273°F
- Keep instruments sharp by occasionally sharpening with sharpening stone.

LiftOut and SlimLift

Usage

The LiftOut and SlimLift are designed for atraumatic extractions. They are inserted into the periodontal space and gently twisted back and forth to cut the PDL's. This saves the tissue and alveolar bone enabling faster healing and implant procedures.



Note!

These instruments are very thin and designed for cutting. Do not elevate with. Excessive force will damage instruments.



Technique

The ergonomics of the handle and non-slip grip enable optimal tactile sensitivity.


1. Insert tip of instrument as far down the periodontal space as possible.
2. Continue working apically while **gently** twisting instrument back and forth.
3. Repeat above from different sides of the tooth to loosen and remove tooth.
4. In cases where the tooth is still attached and more torque is needed for removal, switch to a **TwistOut** instrument for final removal of tooth.



TwistOut

Usage & Technique

The TwistOut is designed for tooth extractions when more force is needed. The TwistOut can be used like an elevator or in a luxating motion.

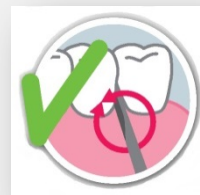

View
LiftOut
Video



View
SlimLift
Video



View
TwistOut
Video



View
Our Web
Page



JS DENTAL
MANUFACTURING, INC.

PO Box 904
Ridgefield, CT 06877
203-438-8832
info@jsdental.com
www.jsdental.com